

SAMi-2 to SAMi-3 Upgrade/Replacement Instructions

If you are a Kit owner (using the SAMi router and iPod), see the SAMi Kit Owners section below.

Otherwise, simply follow the instructions in the Quick Start Guide included with your SAMi-3.

Note:

1. The SAMi-3 camera uses a new app called SAMi3. You cannot use the older SAMi app with the SAMi-3 but you can continue to use it with your SAMi-2 and to access recordings made on the SAMi-2. The SAMi3 app is easily found in the App Store by searching for "SAMi3.
2. The SAMi-3 camera uses the same AC power adapter and bracket as the SAMi-2. If you are replacing your SAMi-2 with the SAMi-3 you can mount it using the SAM-2 bracket and continue to use the SAMi-2 AC adapter.

SAMi Kit Owners:

Because SAMi-3 uses the new SAMi3 app, you cannot use the preinstalled SAMi app on your iPod touch. These instructions will step you through updating your iPod and installing the SAMi3 app.

WARNING: These steps are more complicated than we would like, but this is how Apple does it. Read through the instructions first and do not proceed unless you feel comfortable following them.

You will need an iTunes account, and even though the SAMi3 app is free an iTunes account requires a linked credit card. Set this up first if you don't already have an iTunes account by visiting "appleid.apple.com" using a browser on a computer.

Connect the iPod to your home Wi-Fi network so it can access the internet.

1. Touch the settings (the gear icon at the bottom).
2. Select the Wi-Fi settings (second from the top in the main settings screen).
3. Select your home Wi-Fi network.
4. Enter your password and choose "Join."
5. Confirm that your network is now selected (it should have a check mark).
6. Exit by pressing "Settings" in the upper left.

Now set up your iTunes account.

1. Scroll down until you see "iTunes & App Stores" and then select it.
2. If the Apple ID is not your own Apple ID, select it. Otherwise skip to the next section.
3. If you are signed in to an incorrect Apple ID (such as sami@hipassdesign.com), choose "Sign Out."
4. Select the Apple ID and enter in your own Apple ID.
5. Enter your password.
6. Press "Sign in."

Next, confirm that you are running the latest version of iOS (the iPod operating system).

1. Press "Settings" in the upper left to return to the main settings screen.
2. Scroll up and select "General."
3. Select "Software Update."
4. If there is an update available, select Download and install it. When the iPod finishes downloading the update, it automatically restarts. After it restarts proceed to the next section.
5. If there is not a new software update, simply press the home button to exit settings.

Now install the SAMi3 app.

1. Exit settings by pressing the home button.
2. Click the home button.
3. Select the "App Store"
4. Search for "SAMi3"
5. Install the app. Be sure it is the "SAMi3 Sleep Activity Monitor By HiPass Design LLC".

Finally you need to "forget" your home network and reconnect to the SAMi Router. It is important to "forget" your home network because if you do not, the iPod might connect to it without warning disconnecting the SAMi app from the camera.

1. Select "Settings."
2. Select "Wi-Fi."
3. Select the button to the right of the currently selected network (either a blue forward arrow or "i" in a circle).
4. Select "Forget this Network."
5. Confirm the "Forget."
6. Select "Wi-Fi" in the upper left to return to the list of networks.
7. Now select the "SAMi" network if it has not already been selected for you.
8. Press the home button to exit settings.

You can now remove your SAMi-2 camera and replace it with the new SAMi-3 using the same bracket and AC adapter. Plug in your SAMi-3 and start the SAMi3 app. The SAMi3 app will find the SAMi-3 camera and prompt you to set it up.