




PUREGRAFT 50 REFERENCE GUIDE



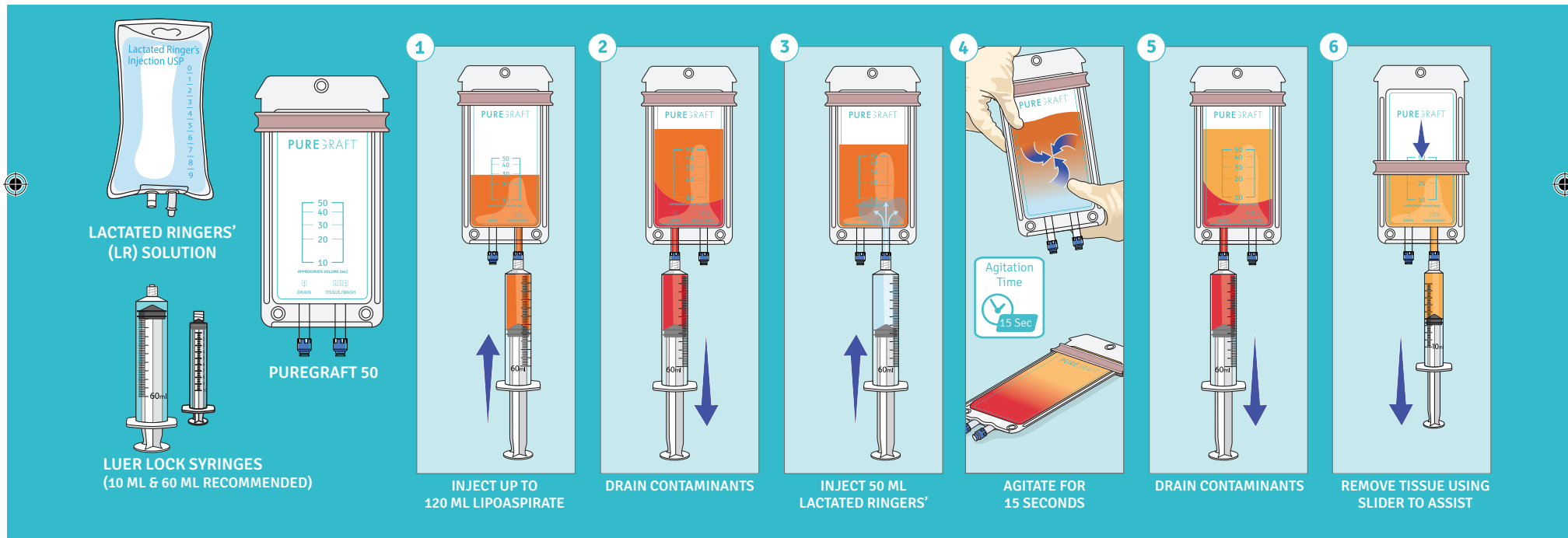

INSTRUCTIONS FOR PUREGRAFT 50

- 1 Inject harvested adipose tissue into Puregraft 50 through “Tissue/Wash” port.
- 2 Connect 60 ml syringe to “Drain” port and remove waste.
- 3 Inject 50 ml Lactated Ringers’ through “Tissue/Wash” port.
- 4 Agitate content for ~15 seconds, lay flat for up to 1 minute and allow for separation.
- 5 Connect 60 ml syringe to “Drain” port and remove waste. Repeat Step 3 through 5 as needed.
- 6 Remove fat for injection through “Tissue/Wash” port using 10 ml syringe. The plastic slider helps to push the tissue towards the port for removal.

HELPFUL TIPS FOR PUREGRAFT 50 USERS

- Overfilling may result in clogging.
- Adipose tissue harvested using cannulae larger than 4 mm may cause clogging.
- To avoid collapsing the bag, hold bag at an upright, angled position and slowly extract all waste fluid through the “Drain” port at approximately 2 mL/second.
- Use a 10 mL or smaller syringe to drain the final waste.
- Before extracting washed graft, discard 0.5 mL of trapped fluid through the “Tissue/Wash” port.
- If the bag clogs or collapses, use a new syringe to insert a small amount of air and LR into the clogged port and agitate the bag.

Puregraft LLC
420 Stevens Ave, Ste 220
Solana Beach, CA 92075, USA

500-037-01 Rev. B

Tel: +1 (858) 348-8050
Email: info@puregraft.com
www.puregraft.com

Puregraft and the Puregraft logo are registered trademarks of Puregraft LLC in the United States and other selected countries.
© 2020 Puregraft LLC. All rights reserved.

