



## Before You Begin

1. Save the cardboard box and place the gel pack flat into the freezer (for at least 4 hours) to ship the sample back to us.
2. Collections must be shipped off to the lab **Monday through Wednesday**. Extra shipping charges will apply for Saturday deliveries (Friday shipments), which may compromise the sample due to the lab being closed over the weekend. **The stool samples for the black topped vials must be collected on two separate days (or at least 12 hours apart)**. All four stool specimens must be shipped together within 5 days of the second collection. We suggest starting the first collection on the weekend.
3. Please check the expiration date on all vials. If any have expired, call 913-341-8949 for replacement vials.
4. Fill out the Test Requisition Form (TRF) completely. Please print clearly. Include your physician or health care practitioner's approval in the space provided on the Test Requisition Form. A physician authorization can be attached to the requisition.
5. Review the collection instructions for your test panel on the following pages prior to collecting any specimens. The Autism Panel has a stool collection. Be aware that the complete sample must be received at the lab **within five days** of the second stool collection.
6. For blood collection, call your phlebotomist and make an appointment for a blood draw.
7. Urine and hair collections are not time-sensitive, however the urine sample must be frozen if not shipped the same day.
8. Consult your physician for specific instructions and before stopping any medications.
9. If you have any questions, please call Customer Service at 913-341-8949 or email CustomerService@GPL4U.com.

## Urine Collection Instructions for GPL-TOX, Glyphosate Test, and Organic Acids Test

1. Please follow the specimen requirements below.
2. Print the patient's full name, time and collection date on the side of the urine cup.
3. Only use containers supplied by GPL to avoid contamination of specimens.
4. Females, please do not collect samples during menstrual cycle.
5. Limit liquid intake starting at 6pm the night prior to collection to avoid sample dilution. Urine must be yellow in color. If the urine is clear, the sample is too diluted. Discard the diluted sample, rinse the collection cup in hot water only (no soap), air dry, then re-collect.
6. After collection, freeze sample for a minimum of 4 hours.
7. Place the urine cup into the biohazard zip-lock bag along with the absorbent packing sheet.
8. Place the frozen gel pack and urine sample into the silver thermobag and place in the cardboard box.



Urine Testing Requirements				
Test	Minimum	Items to Discontinue	Urine Collection	After Collection
GPL-TOX Test	10 mL Urine	None	Collect the first morning urine in the collection cup provided.	Screw lid on tightly.
Glyphosate Test	10 mL Urine	None		
Organic Acids Test	10 mL Urine	Avoid apples, grapes, pears, and cranberries as well as their juices or products 48 hours before collection. Avoid arabinogalactan, echinacea, reishi mushrooms, and ribose supplements for 48 hours before collection.	Collecting sample prior to eating or drinking is preferred.	Freeze urine specimen for a minimum of 4 hours before shipping. Keep in freezer until ready to ship. Ship with frozen gel pack.

# Serum Collection Instructions for IgG Food MAP with Candida + Yeast and Advanced Cholesterol Profile

1. Review the instructions in the Specimen Requirements table below.
2. Please check the expiration date on all vials. If any have expired, call 913-341-8949 for replacement vials.
3. Call your phlebotomist and make an appointment for a morning blood draw.
4. Hydrate 24 to 48 hours prior to your appointment.
5. Take the kit, including all collected specimens, frozen gel pack, and instructions to your appointment.
6. After collections, place frozen vial(s) into the vial sleeve(s) and seal if applicable. Place vial sleeve(s) into the biohazard zip-lock bag. Insert the absorbent sheet (and the frozen gel pack if applicable), and close.
7. Place sealed biohazard zip-lock specimen pouch into the cardboard box.

## Instructions for Phlebotomist


1. Label vial(s) with type of test, patient's full name, birth date, time of collection, and date.
2. Collect blood according to the Specimen Requirements table below and follow the Phlebotomy Instructions included in the Specimen Requirements table.
3. Prepare the sample for shipping. **Samples must be shipped within 24-48 hours after collection.**

## Specimen Requirements

Panel Name	Vial Requirements	Phlebotomy Instructions
Autism Panel	2 Gold SST Vials – No Additive 4 mL Serum 	Hydrate 24-48 hours prior to collection. Allow blood to clot at room temperature, upright for 15-30 minutes. Centrifuge for 15 minutes at 1500g. Separate the serum from the cells. Decant or pipette serum into the 5mL plastic transport vials provided and cap tightly. <b>Freeze sample if not shipping on the day of collection.</b>

## Hair Collection Instructions for Metals Hair Test

- A paper scale is included in this kit to assist you in collecting the proper quantity of hair.
  - Hair that has been dyed, bleached, permed, or otherwise chemically treated is vulnerable to elemental contamination. Therefore, treated hair may not be used for analysis.
  - Use only head or pubic hair for collection and DO NOT mix head hair with pubic hair.
1. Set up the scale as diagrammed. Bend the black tabs down to form a fulcrum and place on a level surface.  

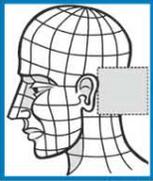  2. Cut samples of hair from the back of the head (see illustration). Using stainless steel scissors, cut hair as close to the scalp as possible. It is best if small amounts of hair are cut from 5 or 6 areas in the back of the head.  

  3. As each piece of hair is cut from the head, save only 1 inch (2.5 cm) of hair closest to the scalp. Cut off and discard the rest. Place the saved hair on the scale.
  4. Keep cutting and adding hair to the scale until it tips. Fill the scale (which is designed to collect about 0.25 grams of hair).
  5. Remove the hair on the scale and put it in the plastic bag.
  6. Fill the scale a second time so that the total amount of hair collected is approximately 1 tablespoon (you will be tipping the scale twice).
  7. Label the plastic bag with the patient's name, the physician's name, and the date collected.
  8. Put the hair into the plastic bag and seal.

# Stool Collection Instructions for *Comprehensive Stool Analysis*

1. The samples for the black topped vials must be collected on two separate days (or at least 12 hours apart). All four stool specimens must be shipped together within 5 days of the second collection. We suggest starting the first collection on the weekend. Vials will have fixative agents in them. Do not pour out liquid. If it spills please contact Customer Service for another kit.
2. Save the cardboard box and place the gel pack flat into the freezer for at least 4 hours to ship back to us.
3. Please check your vials for expiration date. If vials have expired, call Customer Service for replacement vials.
4. Do not allow urine or water to touch the sample as that will contaminate it.
5. Avoid contact of the skin and eyes with the fluid in the vials. For eye contact, flush with water thoroughly for 15 minutes. For skin contact, wash thoroughly with soap and water. For accidental ingestion, contact your physician immediately.
6. If you are unable to produce a stool sample, only the following recommended laxatives can be used:
  - Oral phosphate soda laxatives such as Fleet
  - Magnesium citrate
  - Psyllium fiber
  - Enemas may be given using distilled water only
 Give laxatives 24 hours before any sample collection. Do not use any suppositories or mineral oil.
7. Print the patient's full name, time, and collection date on the vial. You do not have to provide an ID number.
8. Place the white-topped vial and the frozen gel pack into the bubble wrap envelope. Place into the cardboard box.
9. Place the yellow topped vial and both black topped vials into the plastic biohazard zip-lock bag. Next, place the absorbent packing sheet into the biohazard the zip-lock bag with the samples and seal.
10. Place the biohazard bag with the remaining 3 vials into the cardboard box. Seal box.

## Testing Requirements and Guidelines

Vial Type	Minimum Sample Requirement	Items to Discontinue: (Unless otherwise instructed by a doctor)	Collection: Day 1	Collection: Day 2	After Collection
White Top	10 mL stool (¾ full)	Digestive enzymes, antacids, iron supplements, vitamin C over 250 mg, aspirin, anti-inflammatories, and large amounts of meat 48 hrs prior to collection	Collect sample in collection tray. Place stool sample into the vial using spoon attached to the lid. Take multiple samples from different areas of specimen. Fill vial at least ½ full with stool for an accurate test.		Place specimen in freezer.
Yellow Top	5 mL stool (to fill line)			Collect sample in collection tray. Place stool sample into the vial using spoon attached to the lid. Take multiple samples from different areas of specimen.	Shake sample vigorously for about 30 seconds. Store at room temperature.
Black Tops (2)	5 mL stool (to fill line)		Collect sample in collection tray. Place stool sample into the vial using spoon attached to the lid. Take multiple samples from different areas of specimen.	Collect sample in collection tray. Place stool sample into the vial using spoon attached to the lid. Take multiple samples from different areas of specimen.	Shake sample vigorously for about 30 seconds. Store at room temperature.


## Dried Blood Spot (DBS) Instructions – Use the Lancet Provided

1. Using the lancet provided, twist off the yellow tab and pull straight out.
2. Place finger firmly against a hard surface and press lancet to puncture finger. *Using side of fingertip may cause less discomfort.*
3. Wipe off the first blood drop with the same alcohol swab and massage the entire length of the finger as needed to form a large, hanging blood drop.
4. Place a large free-flowing drop of blood on each circle, making sure not to touch collection card with finger. If circle is not filled completely, place additional drops immediately on the same circle.




**The lancet needle retracts immediately after use, leaving the device safe for disposal in a regular trash can. DO NOT return the lancet with sample. Lancets are for one time use only.**

5. If more blood is needed, wipe with alcohol pad again and massage finger to assist blood flow. Be sure to use the second lancet provided as each lancet is for one time use only.
6. Full blood spot circles are required in order to run the test. Completely fill the spot(s). Once dried, DO NOT place additional drops on the same circle.
7. Place the DBS card into the biohazard zip-lock bag then place into cardboard box with the already packaged urine sample.

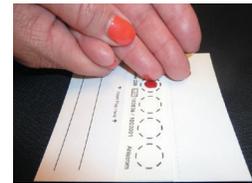

All Dried Blood Spot specimens received at the lab are examined for specimen acceptability. Your sample will be rejected if it is not a satisfactory specimen and a re-collection may be necessary.

## Omega-3 Index Complete Test (1 full dried blood spot)

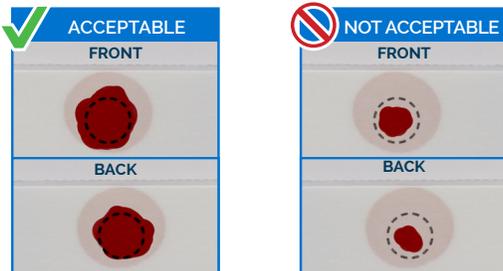

## Preparing and Shipping Specimens

1. Fold and place completed Test Requisition Form in the box and close the lid.
2. Place all urine and blood specimens into cardboard box. The urine and blood samples must be inside the cardboard box, otherwise shipment will be rejected by FedEx.
3. Place cardboard box into the FedEx Clinical Pak bag.
4. Keep record of the shipping/tracking number in the event you would like to track your package.
5. Call FedEx at 1-800-463-3339 to schedule a pickup or take to FedEx location. **Mention you need to schedule a pick-up using a billable stamp. Do not put kits in a FedEx drop box.**