



KVP Alternatives

Recova Shirt™

The Comfortable E-Collar Alternative

Promotes post-surgical healing, preventing pets from agitating sensitive areas. Ideal for treating skin issues and helps reduce separation anxiety.


ITEM #	A*	B**
RSXXS	8.5" - 9.5"	10.5" - 11.5"
RSXS	10" - 11.5"	12" - 13.5"
RSS	10" - 13"	18" - 21"
RSM	14" - 17"	22" - 27"
RSL	18" - 21"	25" - 31"
RSXL	21" - 24"	26" - 32"
RSXXL	23" - 26"	26" - 29"
RSXXXL	29" - 35"	33" - 39"

*** MEASUREMENT A:**

Find the body length by running a tape measure along the pet's spine from base of neck to base of tail.

**** MEASUREMENT B:**

Find the chest circumference by wrapping the tape measure behind the pet's front legs, around the chest.

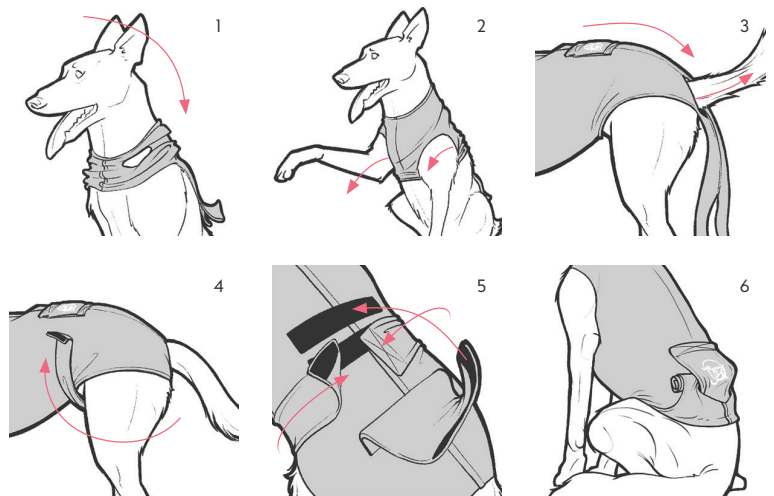

APPLICATION

1. Slide the Recova Shirt over your dog's head.
2. Pull the dog's front legs into the leg holes. The Recova Shirt's fray-resistant material allows for the holes to be cut if a larger opening is needed.
3. Pull the dog's tail through the tail hole.
4. Bring the rear straps underneath the dog's legs and pull them upwards.
5. Pull back the panel to reveal two Velcro® strips. Attach both straps to the bottom strip and close the panel by re-attaching to the top strip.
6. **POTTY MODE:** The lower half of the Recova shirt can easily be rolled or folded back and tucked away for the pet to relieve themselves.