

REF TT1100

REF TT1104

REF TT1106

Instructions for Use



TurtleTub is a Single Infant Use Only Product (with exception below)

TurtleTub may be re-used on multiple patients if disposable liner TT1107 or TT1105 is used.

**LATEX FREE/DEHP FREE
PATENT PENDING**



Manufactured by:
CATAPULT PRODUCTS, LLC
Salt Lake City, UT 84108
385-212-4933
info@turtletubbaby.com
www.turtletubbaby.com



TurtleTub™ Indications for Use:

Intended for infant swaddle bathing.

TurtleTub Instructions For Use:

1. Never leave infant unattended in the bath, not even for a few seconds.
2. Encourage parents to bathe infant.
3. Modify environment as needed to decrease sensory stimuli.
4. Arrange TurtleTub contents for use.
5. For Single-Patient use: Write infant name on label (on side of tub).
6. For Multiple-patient use: Place TurtleTub liner in the tub.
7. Undress infant.
8. If infant has leads, unplug them. The sticky gel portion of the lead can remain on infant. They are easier to remove when wet.
9. Position infant with hands near face and swaddle snugly in fleece blanket.
10. Fill TurtleTub with water until temperature is 101-103 F (38C). Water height should be to infant chest. If using a liner, ensure the liner is smooth (no wrinkles) on the temperature strip.
11. Fill rinse cup with water and place in tub as shown in Figure 1.



Figure 1

12. Place swaddled infant in tub.
13. Use clean 4x4 washcloths with no soap to wash baby's face. Wash one eye at a time moving from inside corner of eye outward. Use clean section of cloth on each eye. Clean outer ear only. Wash neck and behind ears.
14. Gently unswaddle one arm and use baby wash on your hands to wash infant. Be sure to use a firm, but gentle touch. Rinse and re-swaddle. Repeat with all extremities and chest. Remove chest leads. Remember to keep infant snugly swaddled throughout the bath.
15. Gently roll infant to side-lying and clean the infant's back by rubbing on the fleece. Reposition infant to supine.
16. Wash genital area with a 4x4 washcloth.
17. Check temperature strip and add warm water as needed.

18. Wash infant's head last to conserve heat. Use the scalp brush provided and apply enough pressure to bend the bristles. Use water from clean rinse cup to rinse the baby's head.
19. Stand close to the tub with a warm towel (hospital supplied) secured under your arms.
20. Unswaddle infant and lift infant out of tub and into the warm towel. Pat dry making sure infant's head is dry.
21. Place infant in the bed on a warm hospital blanket and wrap infant including the head.
22. Take axillary temperature.
23. Place new leads on infant, if needed.
24. Prepare infant for skin-to-skin holding, feeding, or dress and position for sleep.
25. Cleaning for Single-patient use: Use the pour spout to empty the water from the tub. Wash the TurtleTub with gentle soap and water and dry.
26. Cleaning for Multiple-patient use: Use the pour spout to empty the water out of the tub. Discard the TurtleTub liner. Wash the TurtleTub according to multi-patient product hospital protocol.
27. Wash fleece blanket per hospital protocol for infant wear.

TurtleTub Item Numbers and Contents

| Item # (Qty/case) | Description & Contents | Single-Patient Use System | Multi-Patient Use System | A la Carte Items |
|-------------------|---|---------------------------|--------------------------|------------------|
| TT1100 (10/cs) | TurtleTub Standard Kit - Each kit contains 1-TurtleTub with Integrated Temperature Indicator, 1-Neutral PH Mild Baby Wash, 1-Fleece Blanket, 1-Rinse Cup, 1-Scalp brush, 6-Sterile 4 x 4 Washcloths & Instructions For Use. | X | | |
| TT1101 (50/cs) | TurtleTub Standard Accessory Pack - Each pack comes with 1-Neutral PH Mild Baby Wash, 1-Scalp brush & 6-Sterile 4 x 4 Washcloths. | X | | |
| TT1106 (1/cs) | TurtleTub Starter Kit w/Liner - Each kit contains 5-Bathtubs (each with Integrated Temperature Indicator & Rinse Cup), 10-Fleece Blankets and 25-Accessory Packs w/Liner. | | X | |
| TT1107 (50/cs) | TurtleTub Accessory Pack w/Liner - Each pack contains 1-Disposable Liner, 1-Neutral PH Mild Baby Wash, 1-Scalp brush & 6-sterile 4 x 4 Washcloths | | X | |
| TT1102 (100/cs) | TurtleTub Baby Wash Single Use Packets - 10mL Neutral PH Mild Baby Wash | X | X | X |
| TT1103 (25/cs) | TurtleTub Fleece Blankets | X | X | X |
| TT1104 (10/cs) | TurtleTub Stand Alone Bathtub - (with Integrated Temperature Indicator) | X | X | X |
| TT1105 (100/cs) | TurtleTub Bulk Liners | | X | |
| TT1108 (10/cs) | TurtleTub Mesh Storage Bags | X | X | X |