




Oral Specimen Packaging Instructions



- (1) Please complete:
- Requisition
 - Insurance Information
 - Specimen bottle labels

Label and place specimen container in provided biohazard bag.

(Optional) Remove yellow copy of requisition for your records.

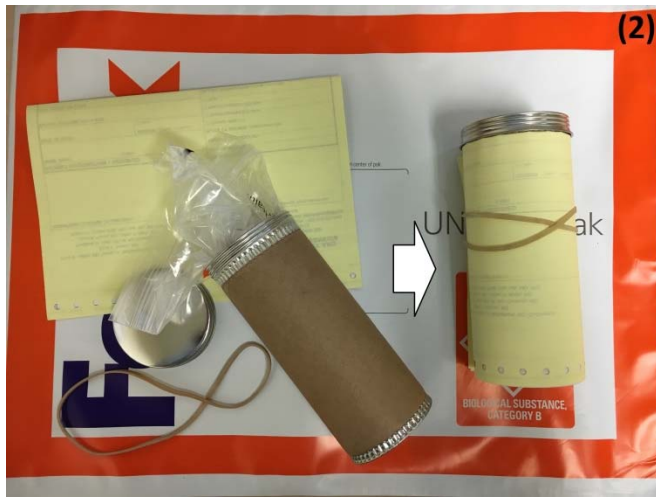

- (2) Please place biohazard bag in provided tube container and screw-on lid.

Attach requisition:

- Fold requisition in half
- Wrap around tube container
- Securing requisition with a rubber band.



- (3) Place wrapped tube container(s) in FedEx Clinical Pak.

Mailing:

- Seal FedEx Clinical Pak**
- Attach FedEx Airbill to package
- Call FedEx for pickup or Drop off at FedEx location