

Supplemental Instructions

Opteryx® Screw Removal Method

▶ When removing the screws from a cervical plate, some key techniques if followed will ensure success.

1. To remove screws, the extraction screwdriver PN CI-060 should be used. Insert the driver shaft into the screw head.



2. Tighten the knurled knob on the handle to thread the inner shaft into the screw's mating thread.



3. Advance the sleeve downward by turning it clockwise.



4. While holding the sleeve, turn the handle counterclockwise to extract the screws.

⚠ Note: IF AT ANY TIME A DEVICE SEEMS TO BE MALFUNCTIONING, REMOVE IT FROM SERVICE AND RETURN IT TO THE MAIN OFFICE FOR REPAIR OR REPLACEMENT.