

How to use EpiPen® and EpiPen® Jr Auto-injectors.

Remove the EpiPen® Auto-Injector from the carrier tube and follow these 2 simple steps:

1



- Hold firmly with orange tip pointing downward.
- Remove blue safety cap by pulling straight up. Do not bend or twist.

2



- Swing and push orange tip firmly into mid-outer thigh until you hear a 'click'.
- Hold on thigh for several seconds.



Built-in needle protection

- When the EpiPen® Auto-injector is removed, the orange needle cover automatically extends to cover the injection needle, ensuring the needle is never exposed.



After administration, patients should seek medical attention immediately or go to the emergency room. For the next 48 hours, patients must stay within close proximity to a healthcare facility or where they can call 911.

For more information visit EpiPen.ca.

EpiPen® and EpiPen® Jr Auto-injectors are indicated for the emergency treatment of anaphylactic reactions in patients who are determined to be at increased risk for anaphylaxis, including individuals with a history of anaphylactic reactions.

This product may not be right for you. Always read and follow the product label.



© 2013 Pfizer Canada Inc., Kirkland, Quebec H9J 2M5 • Toll free: 1-877-EPIPEN1 (1-877-374-7361)
EpiPen®, EpiPen® Jr are registered trademarks of Mylan Inc. licensed exclusively to its wholly-owned affiliate,
Dey Pharma, L.P. of Napa, California USA; sub-licensee, Pfizer Canada Inc., Kirkland, Quebec, H9J 2M5
0000041397



Trusted for over 25 years.