

This is a perfect way to use fabric you no longer need. A fat quarter will give you two with only a bit left.

- 1. Cut fabric 8 by 16 inches.
- 2. Cut interfacing slightly smaller.
 - You can also use dryer sheets and don't have to cut them.
 - Wash & dry them first, and use two per mask.
- 3. Put the interfacing on the wrong side of the fabric. Fold up the long sides about 3/8 inch. This doesn't have to be exact. Just more than 1/4.
- 4. Press it down on the short end and fold up about the same amount twice.
 - This is where the wire for nose grip will fit. If you have pipe cleaners you can sew them in at this point.
 - If you do not have pipe cleaners leave the channel open (removable twist ties will be included with finished masks.)
 - **Note**: Please do not sew in twist ties, they bind up in the wash & stab out.
- 5. After both short ends are folded and stitched fold it in half and press. You will now have a square about 7 by 7.





- 6. Fold three pleats. They don't have to be exact. This step gets much easier with practice. Pin them.
- 7. Cut 7 or 7 1/4 inches of elastic, two per mask.
 - 1/8 inch elastic or round elastic is ideal but 1/4 is good too.
- 8. Insert elastic top and bottom and sew the edge. Back track over the elastic to make sure it's secure. Do the same with the other side.
- 9. Smile! Then make more!





Quality Control: If it's you're first mask or first with a new fabric, try it on and wear it for at least 20 minutes.

- If it is hard to breathe in, try using thinner fabric.
- If it falls off, try making the elastic longer.
- If you can breathe and it doesn't fall off, carry on!



If you are located 20+ Miles away from Bloomington Indiana, please feel free to download this template and use it to start a mask drive in your own community.

More Information & Ongoing Updates: Visit: http://patientphysicaltherapy.com/ and click "COVID-19"

or

http://patientphysicaltherapy.com/bloomington-indiana-fabric-mask-drive-getmeppe/