



# Smartfall - Blue Pendant

## What is it?

Smartfall is an intelligent falls pendant, which allows users to raise an alert by pressing the help button and also automatically detects falls, if enabled. Smartfall can be paired with SmartLink alarms for individuals to live independently.

## Who is it for?

It is suitable for anyone who may be at risk of falling, older people, people working alone, individuals living with limited mobility, long term health conditions, learning difficulties, or those who have been discharged from hospital and require additional support.

## How does it work?

Smartfall allows the user to generate an alarm call from anywhere in the home to the Tunstall 24/7 Customer Care Centre by pressing the help button on the pendant.

If the individual is unable to press the help button, the Smartfall has intelligent sensors which will automatically generate a call for assistance if a fall is detected. The Smartfall pendant is recommended to be worn around the neck for falls detection purposes.

An alarm raised is confirmed via 3 rapid beeps and red LED flashes indicating it is sending a call to the home alarm or mobile device. If a fall is detected then green LED will start to flash and rapid beeps will be generated. In case of a false alarm, the alert can be easily cancelled by tapping the case 3 or more times rapidly.

## Product features

- Manual trigger to raise an alarm
- 24/7 Customer Care Centre availability
- Automatic fall detection alert, if enabled
- LED light confirmation
- Easily cancelled alerts to minimise false alarms
- IP67 Water resistant and shock resistant wearable device
- Replaceable battery and auto low battery reporting
- Compatible with Liberty300 and SmartLink Medi Guardian MKII 4G
- Up to 300m range from the alarm unit

**For more information please visit:**  
**[www.tunstallhealthcare.com.au](http://www.tunstallhealthcare.com.au)**

### Manufacturer

SmartLink International  
A division of Ness Corporation Pty Ltd  
ABN 28 069 984 372  
Unit 4 / 56 Norcal Road  
Nunawading VIC 3131 Australia  
Ph: 03 9875 6400 Fax: 03 9875 6422  
Email: [smartlink@ness.com.au](mailto:smartlink@ness.com.au)  
Website: [www.smartlink.com.au](http://www.smartlink.com.au)

## Specifications

**Product No. 106-383**

### Technical

#### Dimensions:

52x 32 x 12mm (W x H x D)

#### Weight:

25g (0.88oz)

#### Radio frequency:

915 to 928MHz

#### Type:

Crystal Stabilised

#### Radio Range:

Up to 300 metres open air (Subject to environmental conditions)

#### Battery:

CR2430 (3v Lithium cell)

#### Battery life:

12 to 24 months

#### Water Resistance:

IP67 (Up to 1m for 30mins)

#### Bluetooth:

Bluetooth LE (BLE) Upto 20m range.

### Standards

**Radio:** AS/NZS 4268:2012

**Design & Manufacture:** ISO9001:2008

**CE:** Compliant

Tunstall Australasia Pty Ltd is a member of the Tunstall Group.

Contact us on: **AUS t: 1800 603 377 | e: [au.info@tunstall.com](mailto:au.info@tunstall.com)**

**w: [tunstallhealthcare.com.au](http://tunstallhealthcare.com.au)**

**NZ t: 0800 488 678 | e: [nz.info@tunstall.com](mailto:nz.info@tunstall.com) | w: [tunstall.co.nz](http://tunstall.co.nz)**

Our policy of continual development means that product specifications and appearance may change without notice.

Tunstall does not accept any responsibility for any errors and omissions contained within this document.

© 2019 Tunstall Australasia Pty Ltd.