

This documentation provides general instruction for using Radiology Associates of Ocala's **Fuji Synapse PACS (SSL)** client, for referring physicians and hospital use.

System Requirements


The Fuji Synapse PACS (SSL) client is **only supported** on Windows 7 or 8 running Internet Explorer versions 9, 10, or 11.

Any other operating systems or browsers not listed above are **NOT supported**, including but not limited to: *Mac OS, Unix, Linux, Windows XP or lower, Microsoft Edge, Internet Explorer 8 or lower, Google Chrome, Mozilla Firefox, Opera, and Safari.*

How to Install

To install the Fuji Synapse PACS SSL client, use the following steps:

1. In Internet Explorer, open <https://ssl.pacs.raocala.com>
2. You will be prompted to login:

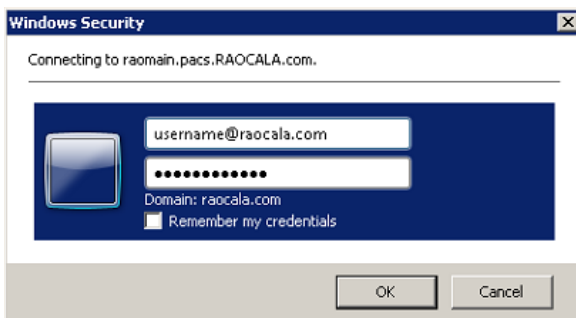

3. Navigate to **Tools > Internet Options > Security (tab) > Trusted Sites**, click the **Sites** button, and add: <https://ssl.pacs.raocala.com> to the list of Trusted Sites.
4. Click the **Synapse Workstation Code Download** button



5. When prompted, click **Run**
6. Allow the installer to run (for 2-5 minutes) and click **Finish** when the install wizard has completed.