


Thank you for purchasing the HydraTherm™. This Quick-Start Guide will help you to get started quickly, but does not replace the Operating Manual. Be sure to read, understand, and practice the operating instructions, warnings, and cautions included in the manual.

INITIAL SET-UP

- Clip shelves onto dividers, then place dividers in tank and arrange packs as desired. See illustrations below for recommendation on how to maximize space for use with 12 packs.

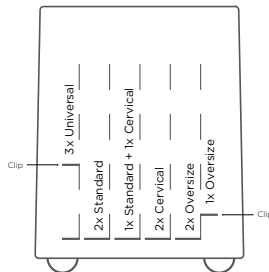

- Find drain hose in rear of device. Ensure the valve* is closed (turned perpendicular to the hose) before filling the device with water.

FILLING WITH WATER

- To fill the device, first pour in **3 cups of tap water**, followed by up to **12 gallons of distilled water**, stopping when the water level has reached 2" from the top of the tank.
NOTE: Exact amount of distilled water varies depending on quantity of packs being used.
NOTE: Distilled water helps to maintain a clean device, but doesn't conduct electricity. For this reason tap water is needed for the Low Water Sensor to operate properly.



POWERING ON THE DEVICE

- Ensure power switch on back of device is in the OFF or O position.
- Connect the included power plug into the rear of the device, and then into the nearest available outlet.
- If using a Deluxe HydraTherm™**, ensure the Heat/Drain switch on the rear of the device is toggled to HEAT.
- Toggle the power switch to the ON or I position. When powered on, the power switch will illuminate, and the digital display on the front of the device will display the current water temperature.

*Standard valve is green/blue. Deluxe valve is gray.

LOW WATER INDICATOR

- Check Temperature Display. If enough tap water was added, the Low Water Indicator should NOT be illuminated at this time. If Low Water Indicator is illuminated, add more tap water until the indicator light turns off.

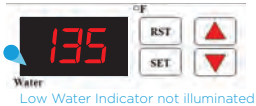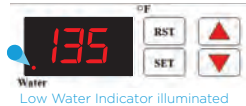

TEMPERATURE SETTING


- Do not exceed 140° F (60° C) for best therapeutic effect, patient comfort, and HydraHeat Pack longevity.
- It can take several hours for the water to reach the Set Temperature. The default Set Temperature is 135°F (57°C), to change the Set Temperature refer to instructions below.

To enter Temperature Set Mode:

- Press “SET” key on control panel, “HSP” will then be displayed.



- Press “SET” key again and the current Set Temperature will display. (e.g. 135°/57°C).

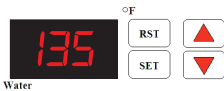

- Press UP or DOWN arrows until desired temperature is reached. (e.g. 140°F/60°C).



- Once desired temperature is displayed, press and release the “SET” button. The new Set Temperature is now saved.
- To exit Temperature Set Mode, press the SET and DOWN arrow together, or the device will exit Temperature Set Mode automatically after 1 minute of inactivity.



WARNING: The scalding temperature of water is 140° F (49° C). Never place hands into the scalding water. Use plastic-coated tongs to retrieve hot pack if necessary.

HYDRAHEAT PACKS

- HydraHeat Packs absorb heat, not water, and provide therapeutic heat up to 45 minutes.
- Placing a towel or pillowcase between between HydraHeat Pack and patient is recommended.
- Terry covers are not recommended when using HydraHeat Packs.

Have questions? Contact Richmar:

Phone: 888.549.4945

Email: customerservice@richmarweb.com

Manufactured for



Compass Health Brands Corp.
Toll Free 1.888.549.4945
6753 Engle Road
Middleburg Heights, OH 44130
richmarweb.com