



When you use a patient lift, you want the security of knowing that the slings you use are durable and well-made.

At Tollos, we're diligent about you and your patients' safety. So we're careful about ensuring the quality of the sling is up to your standards.

Slings are a Class I Medical Device. Tollos takes compliance with all applicable federal regulations very seriously.

We pride ourselves on our quality and your peace of mind.

Repositioning Sling

The Repo Sling is designed for easy repositioning or turning of the patient in bed, assisting with improved compliance with protocols and aiding in the prevention of pressure ulcers.

This sling has a unique strap configuration allowing for easier, faster, and safer connections to the lift.

Repo Sling Features

- Full-length, reclining sling
- Designed for repositioning or turning in bed
- Perfect for floor rescues, even with a backboard in place
- Comfortable, yet durable poly-blended fabrics or mesh
- Color-coded binding allows for quick sizing
- Available in washable reusables or in single-patient-use varieties



General Washing Instructions:

Note: Please refer to the enclosed instructions-for-use packaged with all slings.

All Tollos reusable slings should be laundered according to the instructions on the tag that has been affixed to each sling. The following washing instructions are to be followed for optimum performance to preserve and maintain the quality of Tollos reusable slings:

Spot Cleaning: Most disinfection solutions are suitable for spot cleaning the reusable slings & harnesses. Bleach products can be used in a dilution of 100ppm or 0.01%, however, greater concentrations may cause premature deterioration of the fabrics.

Reusable slings should be laundered between patients.

Washing Instructions:

- Recommended wash cycle: BS 5651:1978 Hospital Laundry Wash (Normal)
- Recommended detergent: ECE Reference Detergent + Sodium Perborate + TAED
- Recommended Wash Cycle: 6 gallons (24 liters) of water at 165°F (74°C) for 15 minutes
Temp not to exceed 185°F (85°C).
- Recommended Rinse Cycle: 4 cycles at 104°F (40°C) for 3 minutes per cycle
- Recommended Dry Cycle: Drying cycle: BS EN ISO 6330:2012 Procedure E
Low Temp, not to exceed 120°F (50°C)

Bleach products may be used during the wash cycle in a dilution of 100ppm or 0.01%.

Do not launder single-patient-use slings.

Sling Guide

Reusable	Reusable Mesh	Single Patient Use	Size	Dimensions	Suitable for:	Binding Color	Safe Working Load
T017LX	T017LXM	T017LX-PSS	Large	40" wide x 79" long (1016 mm x 2007 mm)	174 - 250 lbs	Green	600 lbs/ 272 kg
	T017LXMG		Large	42" wide x 86" long (1067 mm x 2184 mm)	174 - 250 lbs	With elastic corner straps	
T017LXB	T017LXMB	T017LXB-PSS	XL	47" wide x 79" long (1194 mm x 2007 mm)	250 - 400 lbs	Black	1000 lbs/ 454 kg
	T017LXMBG		XL	49" wide x 86" long (1245 mm x 2184 mm)	250 - 400 lbs	With elastic corner straps	
T017LXBE			XXL	66" wide x 79" long (1676 mm x 2007 mm)	400 - 600 lbs	Gray	1000 lbs/ 454 kg

