

Veronikis Vaginal Dilator Set

The Practical Approach to Pain-Free Intercourse

Vaginal dilators are designed to assist women in the treatment of the following symptoms:

- Vaginal tightness
- Painful intercourse due to pelvic floor dysfunction caused by (but not limited to) vaginal atrophy, vaginismus, vulvodinia, vulvar vestibulitis, interstitial cystitis, dyspareunia, vaginal narrowing and similar gynecological conditions.
- Burning sensations related to vaginal penetration
- Vaginal discomfort following childbirth, post-surgical recovery or post radiation adhesions

Care Instructions:

Clean thoroughly before and after use with antibacterial sanitizer, soap and warm water. Be sure to thoroughly rinse all soap residues. Dry with a clean, lint free towel or allow to air dry. Store in a cool, dry place where dilators are not touching each other. Use water-based lubricant only. *Do not use silicone lubricant.*

Important Note:

This information is not intended to substitute for medical treatment or advice. Spotting, light bleeding, or discomfort may be experienced when using vaginal dilators. If it persists or becomes worse, discontinue using the dilators and consult with a physician. **DILATORS ARE FOR INDIVIDUAL USE ONLY.**



- Features .625" (1.6 cm) width
- Dilator length from 1.5 cm to 12.5 cm
- Tapered couplers allow conversion to 1" (2.5cm) width

Product Description

Veronikis Vaginal Dilator Set (8pcs)

Product No.

ARMS-VDS

ARMS Medical provides advanced surgical technologies and tools for pelvic floor issues, including organ prolapse and incontinence. Our goal is to help patients, surgeons and hospitals by optimizing outcomes, maximizing safety and minimizing complications.