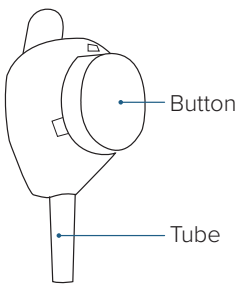
TASSO-SST

VIDEO

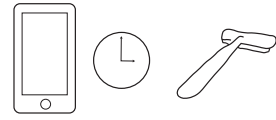
PREPARE

FIRST TIME USERS
Point your phone's camera here to watch:

or go to
www.tassoinc.com/tasso-sst-video

1. BEFORE YOU START:

A. Get a timer.

You'll need it for step 9.

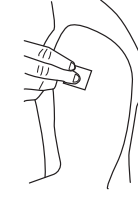
B. Identify collection site.

Refer to step 7 below. If needed, use a razor to remove excess hair from the upper arm.


2. Rub arm quickly and firmly just below the shoulder.

Rub until it's very warm to help your blood flow.

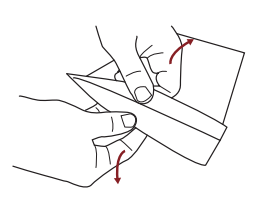
3. Clean arm with alcohol pad.


4. Open device pouch by pulling apart white and clear layers.

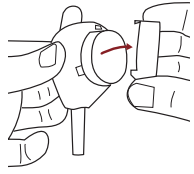COLLECT

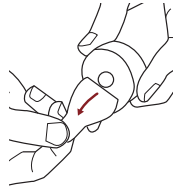5. Remove clear plastic cover over the red button.


6. Peel paper tab behind the red button.

Keep tube pointing down.


7. Stick device to shoulder.

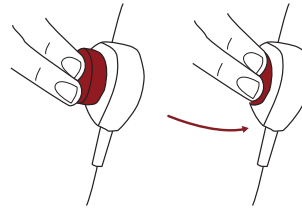Do not remove once it's on.

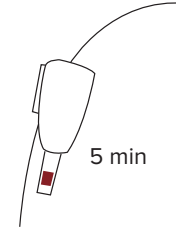
8. Press button firmly until it can't go any farther. Wait two seconds then let go.

9. Start a five minute timer.

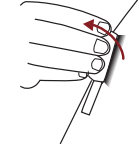
Keep arm at your side. You won't see blood right away.

10. After five minutes or earlier if tube fills, peel off device.


No need to fill the tube completely.

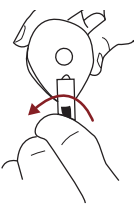RETURN


11. Remove tube by firmly twisting a quarter turn and pulling down.

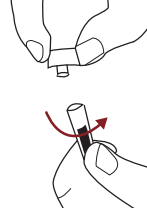This may take a bit of finger strength.


12. Twist cap fully onto the tube.

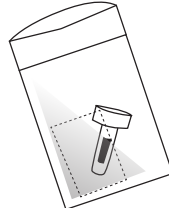Dispose of button and wash hands thoroughly before next step.

13. Place tube in the clear return bag and press to seal.

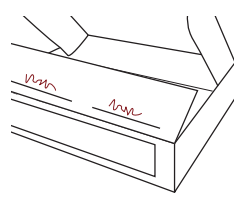
Leave the absorbent pad in the bag.


14. Write collection date and time on the inside flap of the box.

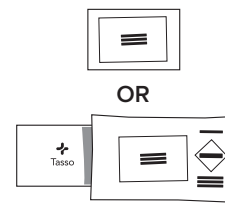15. Place bag in box and seal with the strip.

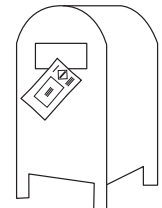16. If grey plastic return bag was included, place box in bag. If no bag was included, return the box.

17. Return bag or box in your outgoing mail or drop it in a US Postal Service drop box.

INTENDED USE
The Tasso-SST is an integrated self-use capillary blood collection device that is intended for the collection of a sample of blood from the upper arm of adults (18 years of age and older) in the home; the system includes materials and instructions for shipping the blood sample back to an accredited laboratory for the analysis of SARS-CoV-2 IgG antibodies, centrifuged upon receipt for IgG and IgM serology measurements.

KIT CONTENTS
Tasso Blood Collection Device
Cap for Collection Tube
Alcohol Wipe
Bandage
Return Pouch with Absorbent Pad
Instructions for Use

SYMBOLS
REF Catalog number
LOT Lot number
IVD For in vitro diagnostic use

STERILE R
Rx ONLY Prescription use only
Sterilized by irradiation

Use by date
Consult instructions for use
This device contains sharps
Do not use if package is damaged
Single use only
Store at 15-30°C

INSTRUCTIONS FOR USE
AGES 4 AND OLDER

TASSO ONDEMAND
TASSO-SST


Contact us for help:
1-408-708-8451

TITLE	
TASSO-SST IFU, IUO	
DWG NO.	REV.
PN0101-01	A4
SHEET 2	OF 9

TASSO-SST

VIDEO

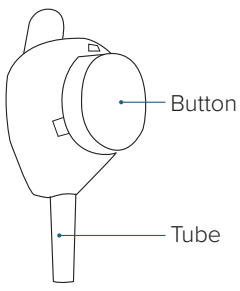
PREPARE


FIRST TIME USERS
Point your phone's camera here to watch:

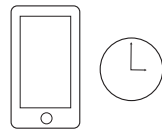
or go to
www.tassoinc.com/tasso-sst-video

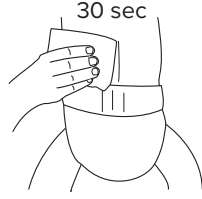
1. Get a timer.
You'll need it in step 9.

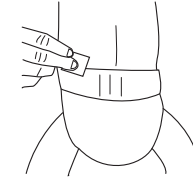2. Hold a cloth warmed under hot water to the side of the lower back just above the diaper line for 30 seconds.

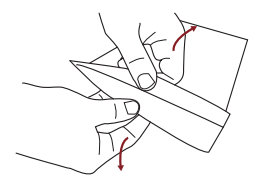
3. Clean warmed area with alcohol pad.


4. Open device pouch by pulling apart white and clear layers.

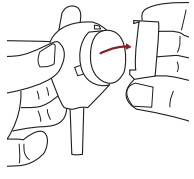
COLLECT

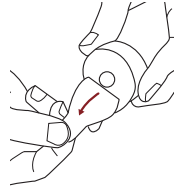
5. Remove clear plastic cover over the red button.

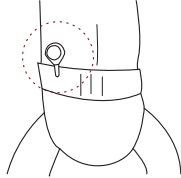
6. Peel paper tab behind the red button.
Keep tube pointing down.

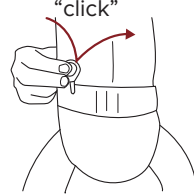
7. Stick device to lower back.
Do not remove once it's on.

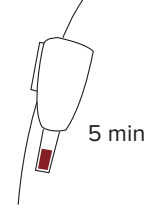8. Press button quickly and firmly until it can't go any farther. Wait two seconds then let go.

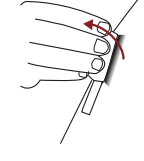
9. Start a five minute timer.
Keep child upright. You won't see blood right away.


10. After five minutes or earlier if tube fills, peel off device.
No need to fill the tube completely.

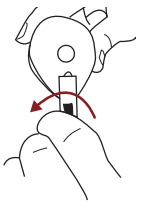
RETURN

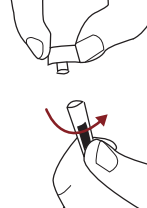
11. Remove tube by firmly twisting a quarter turn and pulling down.
This may take a bit of finger strength.

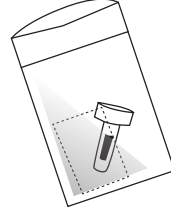
12. Twist cap fully onto the tube.
Dispose of button and wash hands thoroughly before next step.

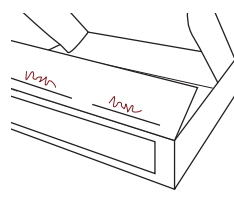
13. Place tube in the clear return bag and press to seal.
Leave the absorbent pad in the bag.

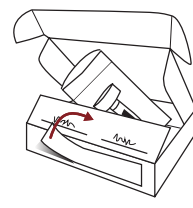
14. Write collection date and time on the inside flap of the box.

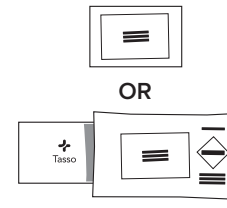15. Place bag in box and seal with the strip.

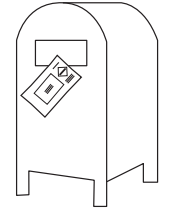16. If grey plastic return bag was included, place box in bag. If no bag was included, return the box.

17. Return bag or box in your outgoing mail or drop it in a US Postal Service drop box.

KIT CONTENTS	
Use by date	⌚
Consult instructions for use	📖
This device contains sharps	⚠️
Do not use if package is damaged	🚫
Single use only	🗑️
Store at 15-30°C	🌡️
Prescription use only	RX ONLY
Sterilized by irradiation	STERILE R

SYMBOLS	
Instructions for Use	REF
Return Pouch with Absorbent Pad	LOT
Bandage	IVD
Alcohol Wipe	
Cap for Collection Tube	
Tasso Blood Collection Device	

INTENDED USE
TASSO-SST is a single use blood collection device that is intended for the self-collection of capillary blood from people ages 3 months and older. The sample is shipped back to a laboratory for analysis, where it is intended to be centrifuged upon receipt for IgG and IgM serology measurements.

WARNINGS
Prescription use only.
Use before the expiration date affixed to the box.
One-time use only.
Keep out of reach of children.
Minor bruising may occur around the sample collection site.
For external use only.
Intended for self-use or use by a healthcare provider.
Use while seated as fainting may occur with any blood sampling procedure.

PN0101-03, A4

Tasso, Inc.
1631 15th Ave. W., Suite 105
Seattle, WA 98119

INSTRUCTIONS FOR USE
AGES 3 AND YOUNGER

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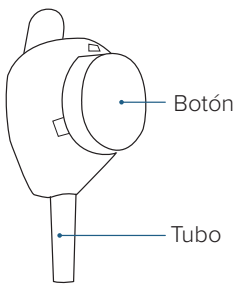
NOTES: (UNLESS OTHERWISE SPECIFIED)

1. SPECIFICATION: IFU MATCHES ARTWORK
 COLOR SPECIFICATIONS
 RED - 7622C
 GREY - 422C
 BLUE - 7706C
 IFU MATCHES FOLDING INSTRUCTION
 PRINTED SIZE: 8.5" x 11"
 FOLDED SIZE: 2.8" x 5.5"
 CURRENT PDF OF ARTWORK MUST BE SENT WITH PO
 PRINTED DOUBLE SIDED, LONG EDGE
2. APPROVED MANUFACTURER: N/A
 APPROVED SUPPLIER: WATER TOWER PRINTING
 1961 S. SPARTAN ST
 GILBERT, AZ 85233
- ALTERNATE SUPPLIER: PRINT FOCUS
 2424 S. 21ST STREET
 PHOENIX, AZ 85034
- IFU MAY BE PRINTED AT TASSO HQ
3. INSPECTION REQUIREMENTS: VERIFY ARTWORK
 VERIFY PRINTING QUANTITY
 VERIFY LEGIBILITY
 VERIFY FOLDED CORRECTLY
 VERIFY PART NUMBER AND REVISION LEVEL
4. PACKAGING/ STORAGE REQUIREMENTS: N/A

DASH NO.	Manufacturer Part No.	Supplier Part No.
-01	Tasso-SST IFU, Adult, IUO, English	N/A
-02	Tasso-SST IFU, Adult, IUO, Spanish	N/A
-03	Tasso-SST IFU, Pediatric, IUO, English	N/A
-04	Tasso-SST IFU, Pediatric, IUO, Spanish	N/A

UNLESS OTHERWISE SPECIFIED

DIMENSIONING AND TOLERANCING
 PER ASME Y14.5M-1994
 ALL DIMENSIONS ARE IN INCHES

TOLERANCES:

- .X = ± .1
 .XX = ± .01
 .XXX = ± .005
 .XXXX = ± .0005
 = ± 1°
 SURF FINISH = 32/√

TASSO

1631 15th Avenue West, Ste 105
 Seattle, WA 98119


The information contained in this drawing is the sole property of Tasso Inc. Any reproduction in part or whole without the written permission of Tasso Inc is Prohibited.

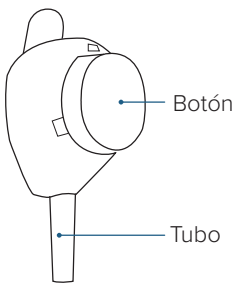DWN	YAM	4/24/2020	TITLE			
CHK	-	-	TASSO-SST IFU, IUO			
ENG	-	-	SIZE	DWG NO.		REV.
MFG	-	-	A	PN0101		A4
QA	-	-				
APPD	-	-				
DO NOT SCALE DWG			SCALE:	N/A	SHEET 1 OF	9

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